

PARAMETERS FOR THE OK-512 DIGITAL CONTROL OF PORTABLE COLD STORAGE UNIT

THE UNIT IS JUST ADJUSTED AND ONLY SET SHOULD BE VARIED IN THE CASE THAT IS WANTED TO WORK WITH A DIFFERENT TEMPERATURE SCHEDULED, WHICH BY DEFAULT IS 16 ° C

GREEN (temperature programmed by the user)

RED (current temperature of the unit)

The desired adjustment value programming:

1- Press **SET** and release

SP1 will be displayed intermittent and their corresponding values currently scheduled.

2- Click the arrows to **move up** or **down** to the desired value.

3- Finally double-click **SET** to confirm the value.

PROGRAMMING DIGITAL CONTROL IN CASE THAT YOU HAVE TO ACCESS TO ANY SPECIAL APPLICATION OR MODIFICATIONS.

To access to a modification of these features an access code we will needed to enter in programming. Please, contact with Distributor to support.